PULSE OXIMETER

Wat betekent de meting?

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PULSE OXIMETER

What does the measurement mean?

A normal saturation value (SpO₂) is between 95-100%. A value below 90% can indicate health problems. In general, the resting heart rate for adults is 60-100 BPM. Consult your physician to determine a diagnosis if the SpO₂ and / or heart rate measurement deviates from the reference values.

DESCRIPTION

The Sinji Pulse Oximeter is a portable device with which the oxygen saturation and pulse (heart rate) can be measured via the finger. Thanks to the advanced DSP algorithm, measurements are very accurate. This oximeter is easy to use and suitable for consumers and professionals.

IN THE BOX

1x Sinji Pulse Oximeter
1x Cord
1x Manual

TECHNICAL SPECIFICATIONS

Model	Sinji Pulse Oximeter
Display	system OLED
Power supply	2x AAA batteries 1.5V
Measurement range	Oxygen saturation: 70% - 100%, pulse: 30 - 254 bpm
Accuracy	Oxygen Saturation: $\pm 3\%$, pulse ± 2 BPM
Automatically off	after 8 seconds
Dimensions	5.8 x 3.6 x 3.3 cm
Weight	50 g

INSERT BATTERIES

Before use, insert two AAA batteries into the device. Open the battery compartment and observe the polarity as indicated in the battery compartment. Close the battery compartment after insertion.

ATTACHING THE CORD

Attach the supplied carrying loop by putting it through the hole.

INSTRUCTIONS

1. Open the finger hole by squeezing the part marked "push".
2. Place one of your fingers into the rubber finger hole.
3. Press the start button to start the measurement.
4. Keep your finger and body as still as possible during the measurement for best results.
5. After a short while, the pulse rate and measured oxygen saturation will be indicated on the display.
6. Remove your finger from the finger holes and the device will automatically turn off after 8 seconds. Clean the rubber part after use.

SAFETY TIPS

- This device should not be used in the vicinity of magnetic resonance imaging (MRT) or computer tomographs (CT).
- Do not use the oximeter in the vicinity of explosive, resp. flammable materials - risk of explosion!
- The device is not suitable for continuous monitoring of oxygen saturation in the blood
- Finger and device must be clean so that a problem-free measurement can be taken.
- Inaccurate measurements can also arise in case of:
 - Dysfunctional hemoglobin or low hemoglobin level
 - Use of intravascular dyes - Bright environment - greater hand or body movements
 - Use of electrosurgical interferences and high frequency defibrillators
 - Venous pulsation artifact
 - Simultaneous use of blood pressure monitor cuffs, catheters or intravascular ports
 - Patients with high blood pressure, vasoconstriction, anemia or hypothermia
 - Cardiac arrest and shock states
 - Artificial fingernails
 - Circulation disorders
- The oximeter is only an additional aid in assessing the patient's situation.
- Electrosurgical instruments can impair the performance of the device
- The health situation can only be determined when a doctor has more clinical, resp. conduct professional investigations
- During prolonged use or after use on each patient, the measuring point should be changed regularly. The measurement point should be changed every 4 hours, checking the skin integrity and circulatory status of the patient.

CLEANING

Remove the batteries before cleaning the device. Never use aggressive cleaning agents or hard brushes. Clean the device with a soft cloth slightly dampened with isopropyl alcohol. Moisture must not penetrate into the device. Do not use the device again until it is completely dry.

PROBLEMS AND SOLUTIONS

Problem	Cause	Solution
Oxygen saturation or pulse does not display properly	<ol style="list-style-type: none">1. The finger is not properly inserted in the finger hole.2. The user's oxygen saturation is too low and cannot be measured.	<ol style="list-style-type: none">1. Place the finger correctly in the finger hole and try again.2. Verify that the device is working properly by taking a measurement on another person.
Oxygen saturation or pulse is unstable	<ol style="list-style-type: none">1. Finger is not far enough in the device.2. The finger or body is moving too much.	<ol style="list-style-type: none">1. Place the finger correctly in the finger hole and try again.2. Try to keep your body still.
The device does not turn on	<ol style="list-style-type: none">1. The batteries are empty.2. The batteries are not properly inserted in the device.3. The device is damaged.	<ol style="list-style-type: none">1. Replace the batteries2. Check if the batteries are installed correctly3. Contact customer service
The backlight of the screen suddenly turns off	<ol style="list-style-type: none">1. If no finger is detected, the device will turn off automatically after 8 seconds.2. The battery has insufficient power.	<ol style="list-style-type: none">1. Place your finger in the finger hole and try again2. Replace the batteries

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